

Virtual reality visual rehabilitation solution

Your orthoptist has programmed a visual rehabilitation session specially for you using the Eyesoft EMAA Home solution.



Find our tutorial video to get started with the headset here →



On/Off Button

→ Turn on the headset

Press and hold the On/Off button on top of the headset for **5 seconds**.

→ Turn off the headset

Press and hold the On/Off button on top of the headset for **5 seconds until the LED flashes blue**.



WiFi Connection

To connect the headset to WiFi, choose "Configure", select your network, then enter the password using the central button on the right side of the headset. This step allows you to retrieve your sessions programmed by your orthoptist.



Start a session

To access the sessions programmed by your orthoptist, select "My sessions", then choose the one to perform.



Time Limit

The following message may appear: *"Warning, you have exceeded the number of minutes... Please try again later."*

This limit is activated when the headset was not turned off correctly or remained on standby.

You can contact your orthoptist to have this limit reset.



To ensure you receive the best possible support, your orthoptist is available for any questions. Please read the user guide carefully before first use.