

Find our tutorial videos to get started with the headset here →



On/Off Button

→ Turn on the headset

Press and hold the On/Off button on top of the headset for **5 seconds**.

→ Turn off the headset

Press and hold the On/Off button on top of the headset for **5 seconds until the LED flashes blue**.



Headset Positioning

After placing the headset on the patient's head, they can adjust the position (tilt, height and tightness) to obtain a clear view of the head displayed in the headset. If the image remains blurry, the headset needs to be repositioned.



Calibration

For each new patient and at the beginning of each session, calibration is essential. To do this, ask the patient to aim at "Calibrate" or click "Calibrate" from your Eyesoft interface.



VR Realignment

To select settings, click the central button **1**

If you don't see the menu in front of you in the headset, you can realign the VR environment by pressing and holding for 3 seconds the button **2** at the bottom right of the headset. Or by clicking this button  located at the bottom right of the exercise tracking page.

 Please read the user guide carefully before first use.